

Turn Plate for the NanoVi™ Device


1. The NanoVi™ Turn Plate (Article #: 4380-00) is custom-fitted for NanoVi™ devices.

The Turn Plate clicks into eight different positions to prevent further rotation when the NanoVi™ device is being used.



2. To attach the Turn Plate; place the NanoVi™ device face down on a soft surface.

Make sure that the bottom of the device is clean, if not, clean it with window cleaner.

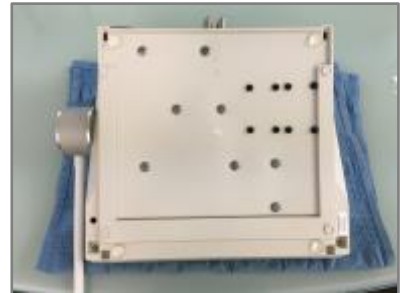

3. The Turn Plate comes with five round white adhesive Velcro dots already attached to the black Velcro blocks.

Remove the plastic from the adhesive but leave the white Velcro dots attached to the black Velcro blocks.



4. Turn the Turn Plate so the Velcro is towards the device, but don't let the adhesive touch the bottom of the device.

Adjust the position according to the picture with the flat side of the Turn Plate in line with the backside of the device. Center the Turn Plate so the left and right sides align with the sides of the device. Press the Turn Plate firmly on to the bottom of the device.



5. Place the NanoVi™ device on a solid surface. The Turn Plate allows the NanoVi™ device to be rotated 360°. It has eight stop positions to keep the device from rotating unintentionally.

To transport the NanoVi™ device in the Transportation Case; the Turn Plate must be removed. Simply pull it off. The round, white Velcro dots will stay on the bottom of the device. The Turn Plate can be easily reinstalled by aligning it to the bottom of the device so the Velcro reconnects.

